**Rotaid** AED Cabinets

# **SOLID PLUS** HEAT LED

INSTALLATIEHANDLEIDING • GUIDE D'INSTALLATION • MONTAGEANLEITUNG  
INSTALLATION MANUAL • MANUALE DI INSTALAZIONE • MANUAL DE  
INSTRUCCIONES • INSTRUKCJA MONTAŻU • INSTALLATIONSMANUAL

# Parts-list

## This Rotaid package contains:

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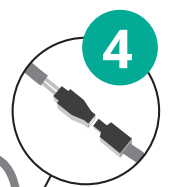
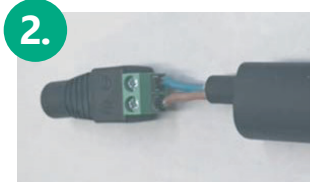
# ROTAID SOLID PLUS HEAT LED English instructions

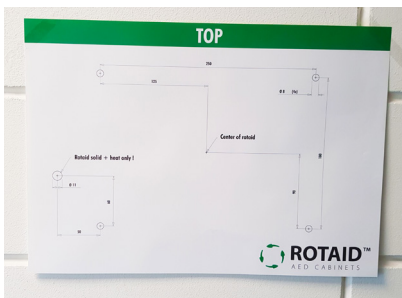
**PLEASE NOTE: Only a certified company or electrician should install the cabinet because of an electrical installation. The heating is not ready for use yet!**

**1.** Determine the location where you wish to mount the Rotaid. Make sure there are no irregularities present at the desired location. Please take the fire safety and evacuation regulations into account. If mounted on a stone or concrete wall, please use the wall anchors prior to fixation.

**To ensure the defibrillator stays within its operational temperature range during the whole year, the Rotaid should be located in the shade at all times.**

**2.** Use the paper mould to mark the holes for mounting the back plate, 4 in total. Please also mark the additional hole to house the power cable.

**3.** On the bottom-left of the back plate an eye is located. Use an awl or screwdriver to puncture the sealing plug. Run the whole cable of the heating element through the hole.

**Do not attach the heating element to the back plate yet!**

**4.** Attach the back plate by using the four screws. Next, remove the red layer on the back of the heating device and fixate the heating system to the backplate. When correctly placed, the control unit on the heating device is at the bottom left side.

**5.** With the back plate fixed to the wall, now use an Allen key (5mm) to firmly attach the nylon belt to the back plate.

**6.** After attaching the belt and cover, now please mount both long bolts in the indicated holes. Mount these by hand via a screwdriver and do not overtighten. You can now hang the AED via its handle on the bolts. Make sure the backside of the AED is as close as possible to the heating system.

**7.** You can now place the AED into the cabinet by using its handle or carrying case handle. Please rotate the cover on the back plate by starting slightly counterclockwise and rotate to the left until the cover reaches the stop. When correctly placed, the hook on the bottom of the cabinet falls through the cover.

Continue to **instruction sheet A.**

# ROTAID SOLID PLUS HEAT LED Instruction sheet A.

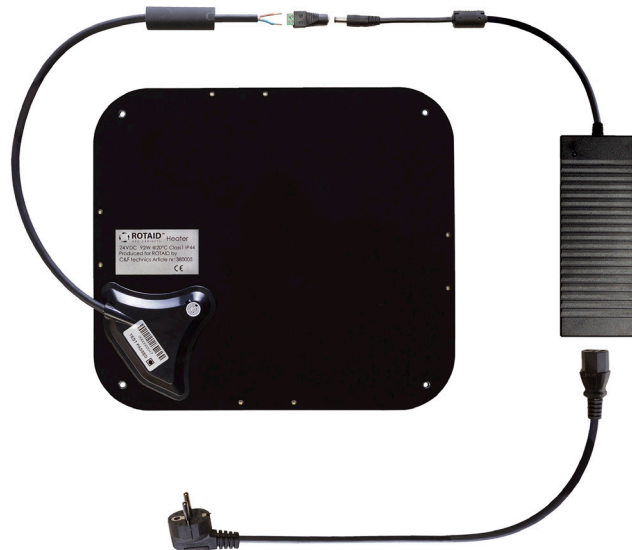
## WATCH OUT! ATTENTION!

Follow the instructions!  
\*Possible chance of damaging the product.


+ Brown - Blue

### Attach a connector plug to the cable.

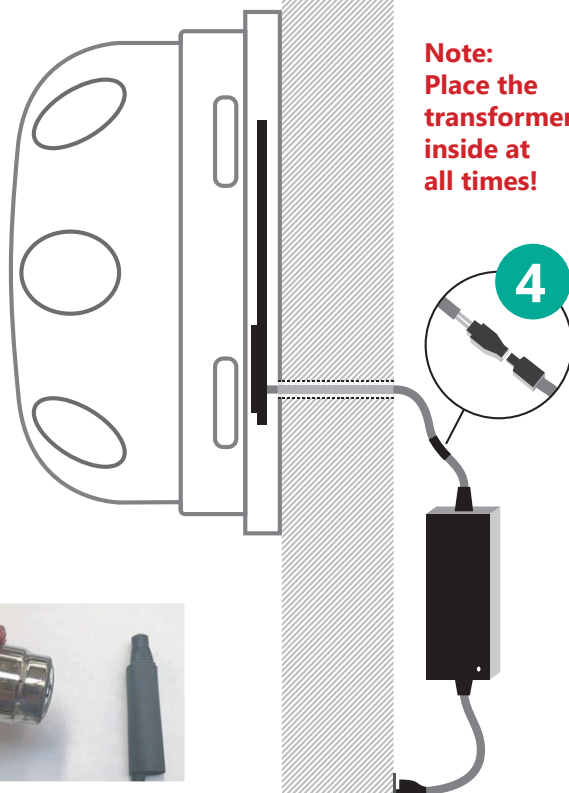1. Cut the cable to length and strip the outer cable (about 2 centimeters).
2. Remove 5 mm of the brown and blue isolation to unveil the copper wire. Cut off the green-yellow cable completely (if present).
3. Place the crimp sleeve insulator over the connection wire. The cable can also be insulated with tape (after connection).
4. Connect the wires to the connector.
5. Insulate the wires via the sleeve insulator or tape.
6. Now connect the plug to the connector that is attached to the transformer. Plug the power cable into the wall outlet.
7. Immediately after installation, the heating system will go into start-up mode and perform a full system check. Part of this check is that the heater will go to its maximum output for a limited time. As a result of this, the heating mat will become hot. When the system check is successfully finished, the LEDs will be on constantly and the system will go into stand-by mode and its surface temperature will drop. When the system check fails, the LEDs will start to blink. You can then contact your Rotaid partner for further assistance.

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